



Start the program GS remote (dongle has to be on).

Abb. 1

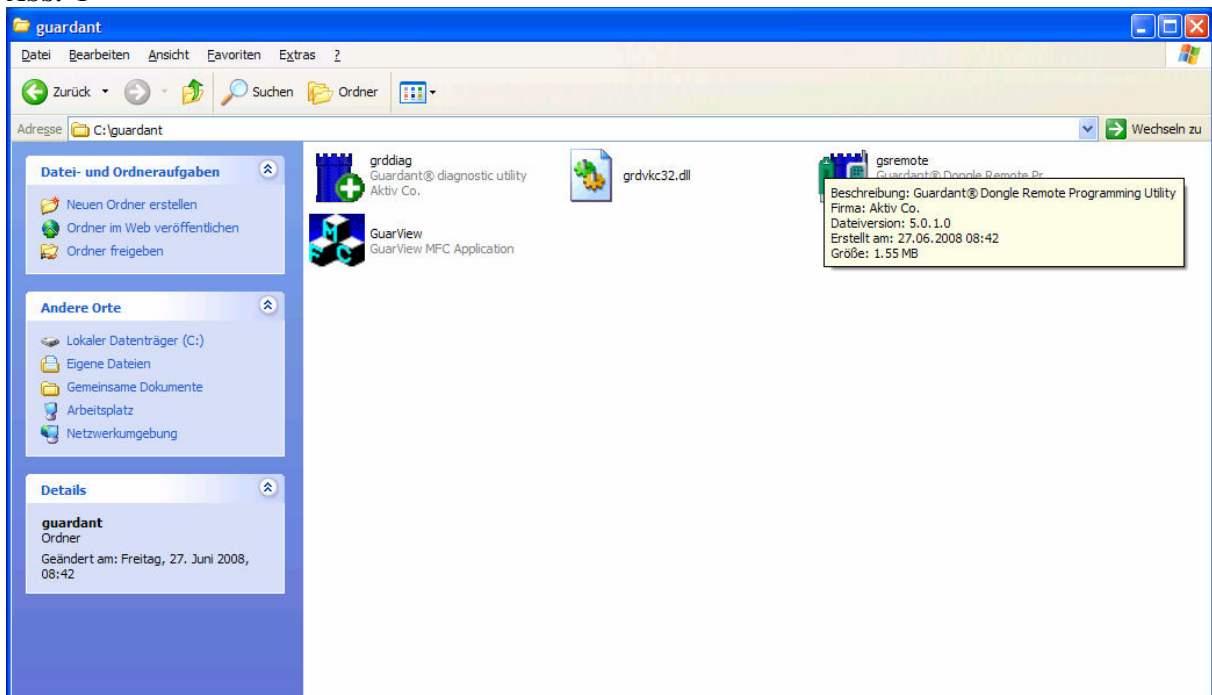

This picture will appear:


Abb. 2

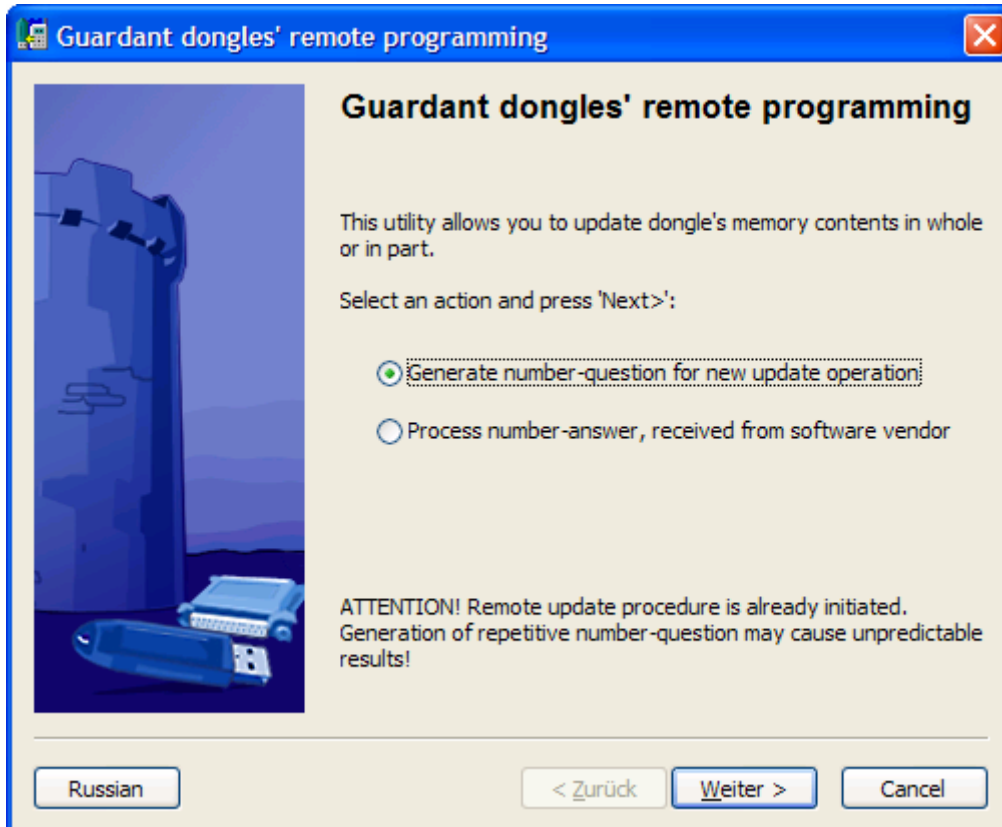

Press generate number question for new update operation (and next):


This window will appear:

Abb. 3

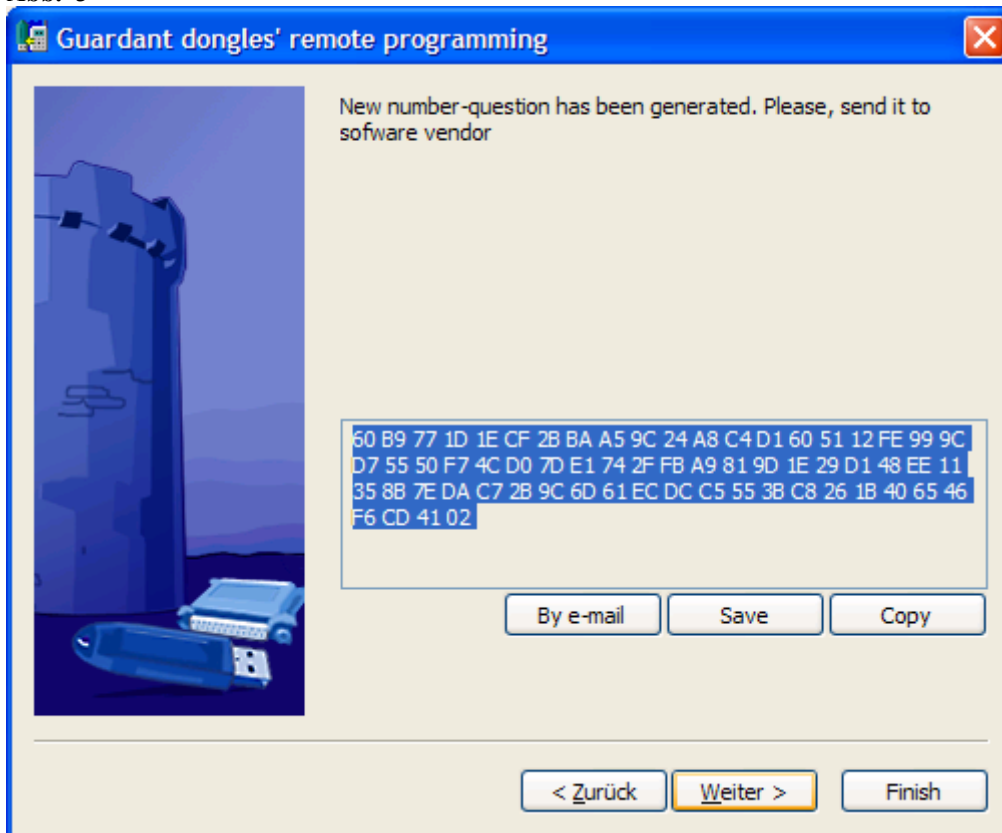

Save this number by click on save button:


Abb. 4

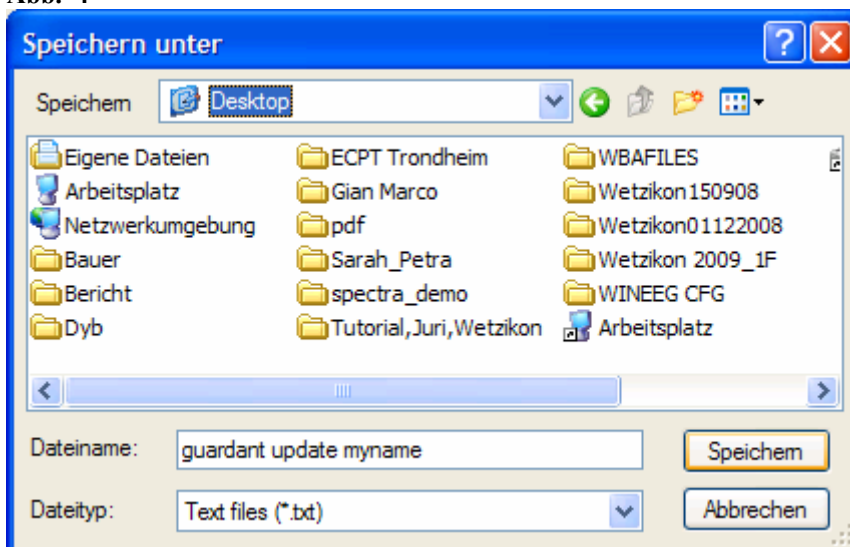

Send this file to [update@hbimed.ch](mailto:update@hbimed.ch) by e-mail (or to: [andreas\\_mueller@swissonline.ch](mailto:andreas_mueller@swissonline.ch))


Number answer (when you have got the new number):

Abb. 5

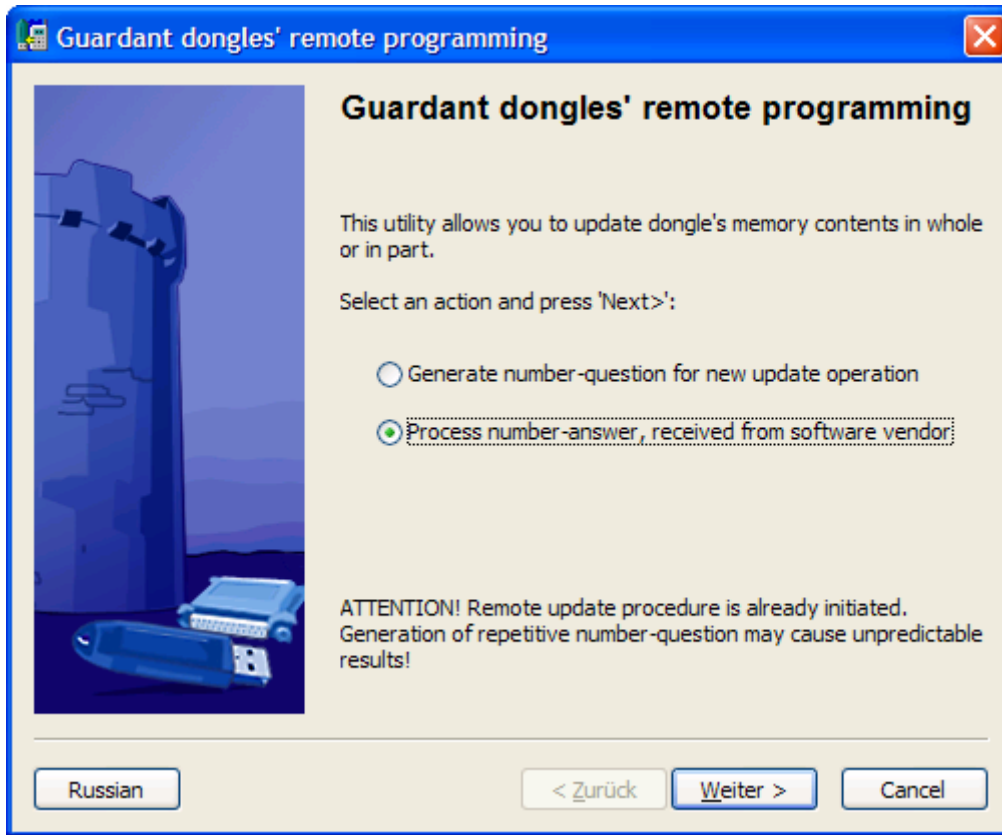

Click to process number answer


Abb. 6

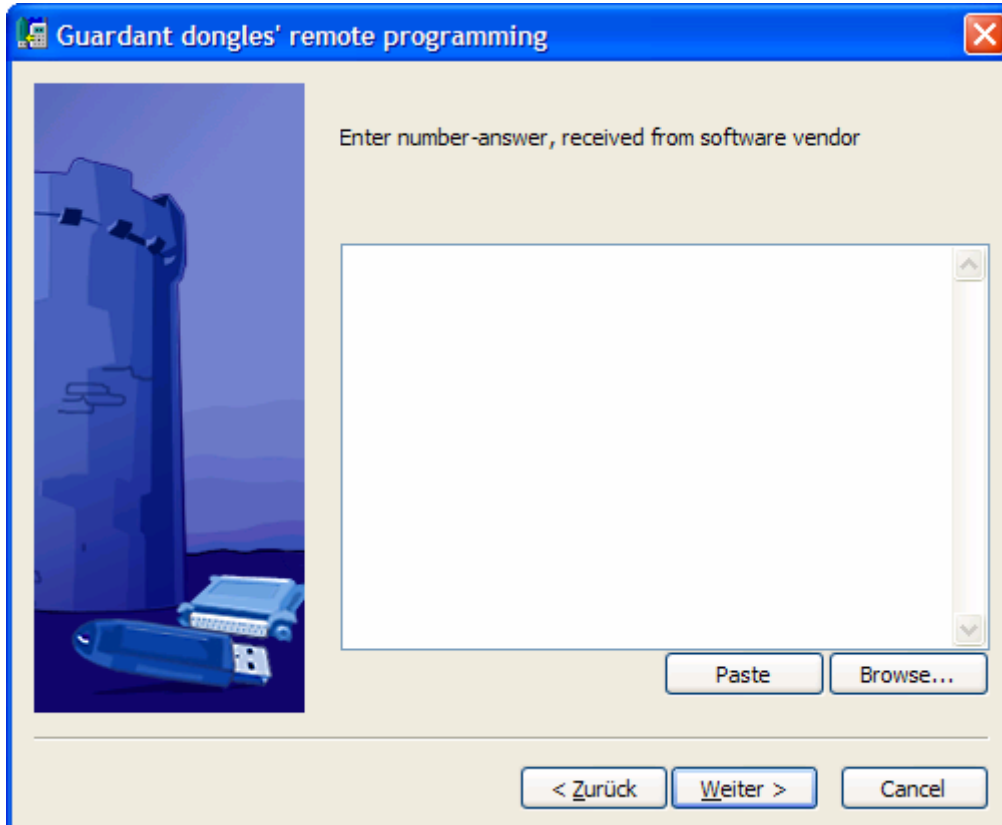

Load number answer file: Click to browse


Update myname

Abb. 7

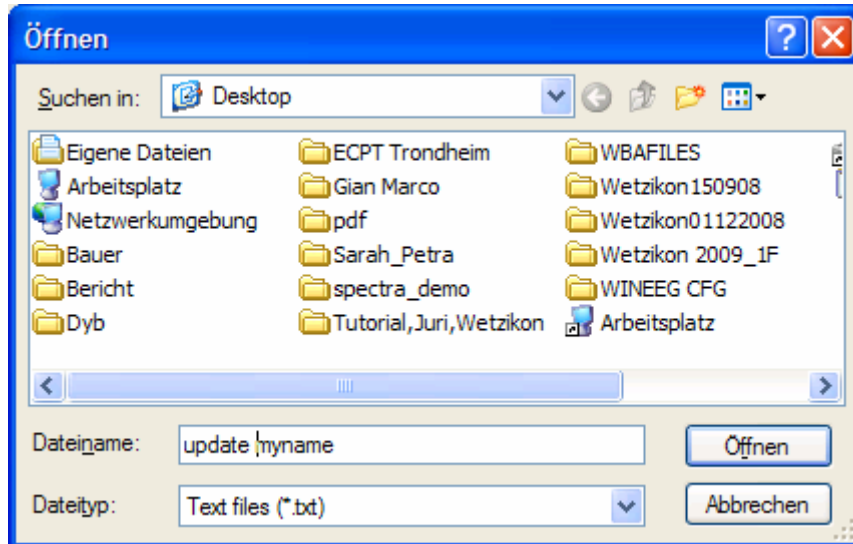

Open, then the window appear:

Abb. 8

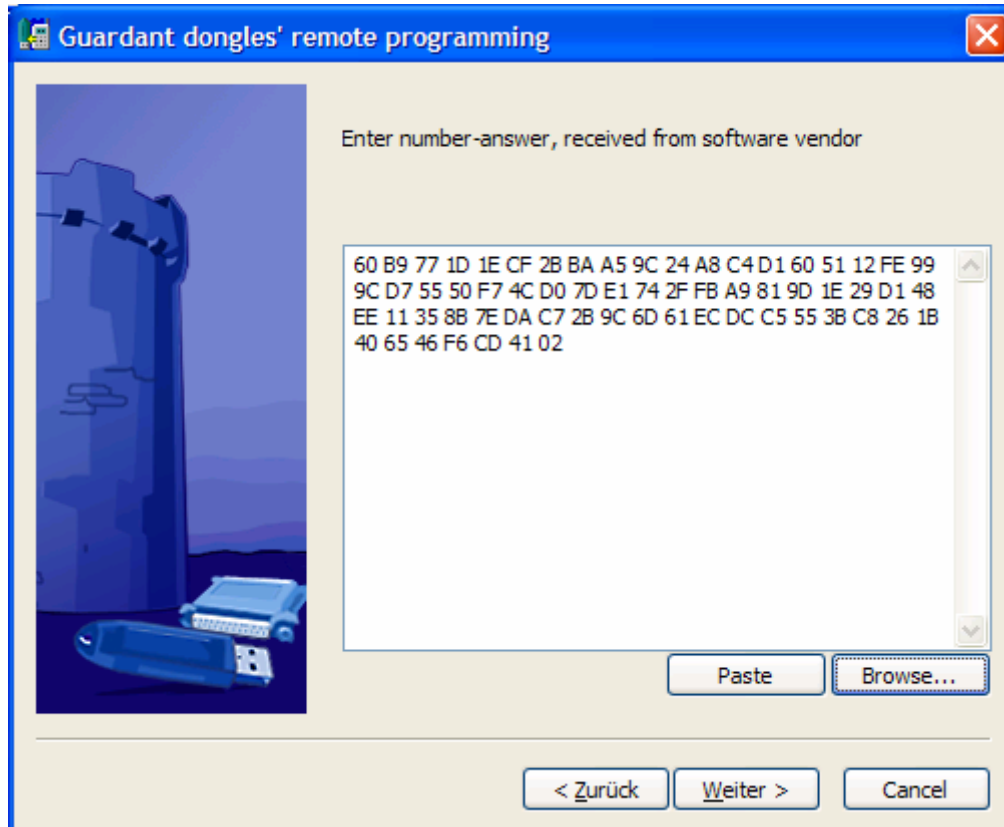

And next.

If the programm did not update your dongle successfully contact hbimed.